

Orthopedics Extension



Ortho extension with cart,CF

During the orthopedic traction surgery, it is used to pull the femur to meet the need of x-rays; the device is made of highly-transmissible carbon fiber materials to improve image qualities.



Part No.: 115-066779-00/115-029436-00

Accessory Name :

Ortho extension w/ cart,CF,comfort

Ortho extension w/ cart,CF,basic

Application : To provide leg traction during orthopedic surgery and access to X-ray scanning

Compatible with : HyBase8300/8500


Ortho extension w/ cart, III



Countertraction post



Traction boot
Traction boot, comfort

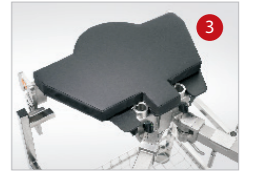

Pelvis plate



Screw tension device



Foot plate support



Adapter



Orthopedics Extension

Part No.: 115-009425-00/115-066777-00

Accessory Name :


Ortho extension w/ cart, III, basic
Ortho extension w/ cart, III, comfort

Application : To provide leg traction during orthopedic surgery and access to X-ray scanning

Compatible with: HyBase6100/3000

Orthopedics Extension Installation Steps

Step 1

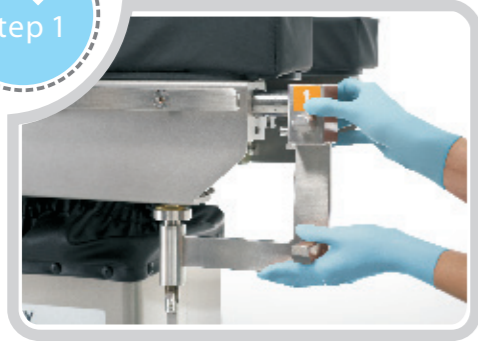

Connect the adapter with table

Step 2



Fix the screw

Step 4



Press the "up" button of the remote control

Step 3



Push the transport cart

Step 5



Remove the transport cart

Step 6



Install the supports


Laser scale can help the surgeon adjust the patient for the precise positioning.



The elongation bar can be adjustable to fit the different operation environment.



Triangle support board can give patient the most comfortable place during the operation.

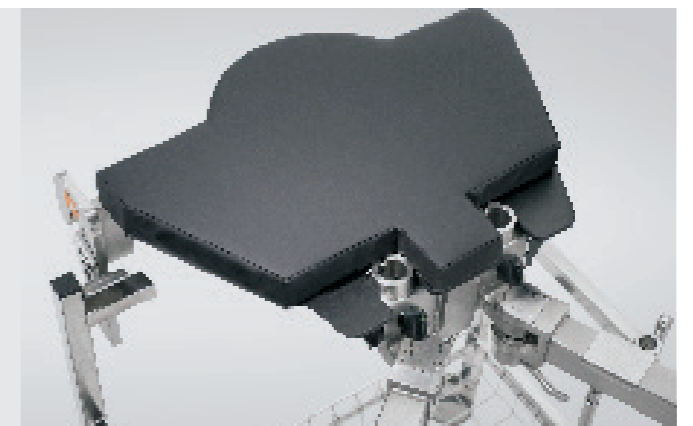

Orthopedics Extension

Other Accessories



Traction stirrup clamp



Countertraction post for femur



Foot support assembly



Universal support for positioning the lower leg for transcondyle wire extension



Traction stirrup clamp



Countertraction post for tibia and fibula



Screw extension device adaptor



Leg Support



Lateral Femoral Extension Assembly

Part No.: 115-007057-00

Accessory Name : Lateral femoral extension assembly

Application : To provide the shank and access to X-ray scanning



Tibia Extension Assembly

Part No.: 115-007055-00

Accessory Name : Tibia extension assembly

Application : To provide tibia and fibula traction

Orthopedics Extension

Other Accessories




Leg holder for Ortho extension device,SS Leg holder for Ortho extension device,CF

Part No.: 115-047827-00 / 115-047828-00

Accessory Name : Support the leg,CF (optional),used withOrtho extension device

Application : Support the injured leg during the orthopedic operation

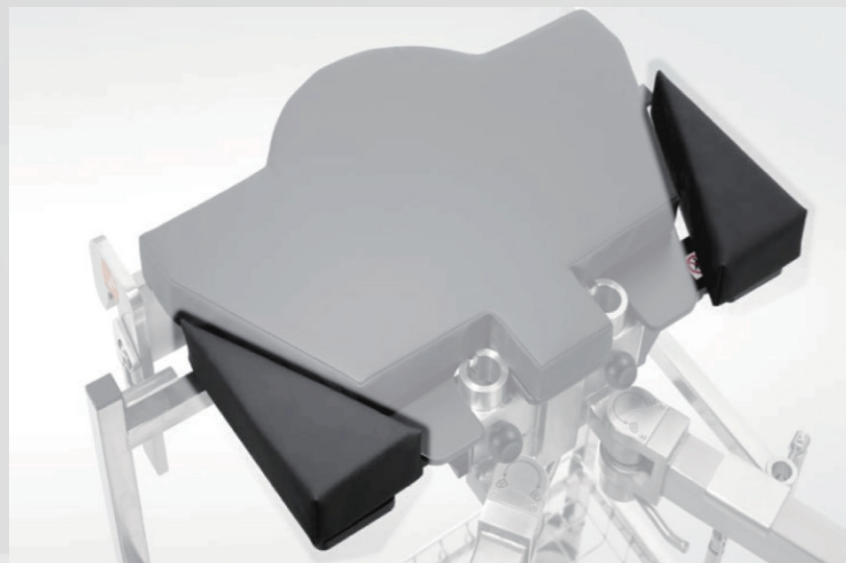

Pelvis width extender, a pair

Part No.: 115-011817-00

Accessory Name : Triangle support board, 2 pieces.

Application : It is used to widen the pelvis plate.

Ortho extension w/ cart, IV



Part No.: 115-066781-00/115-033409-00

Accessory Name :

Ortho extension w/ cart, IV, comfort

Ortho extension w/ cart, IV, basic

Application : To provide leg traction during orthopedic surgery and access to X-ray scanning

Compatible with: HyBase8300/8500

Ortho extension w/ cart, Uni



Part No.: 115-066775-00 /115-033410-00

Accessory Name :

Ortho extension w/ cart,Uni,comfort

Ortho extension w/ cart,Uni,basic

Application : To provide leg traction during orthopedic surgery and access to X-ray scanning

Compatible with: UniBase30